THUASNE

MOBIDERM autofit

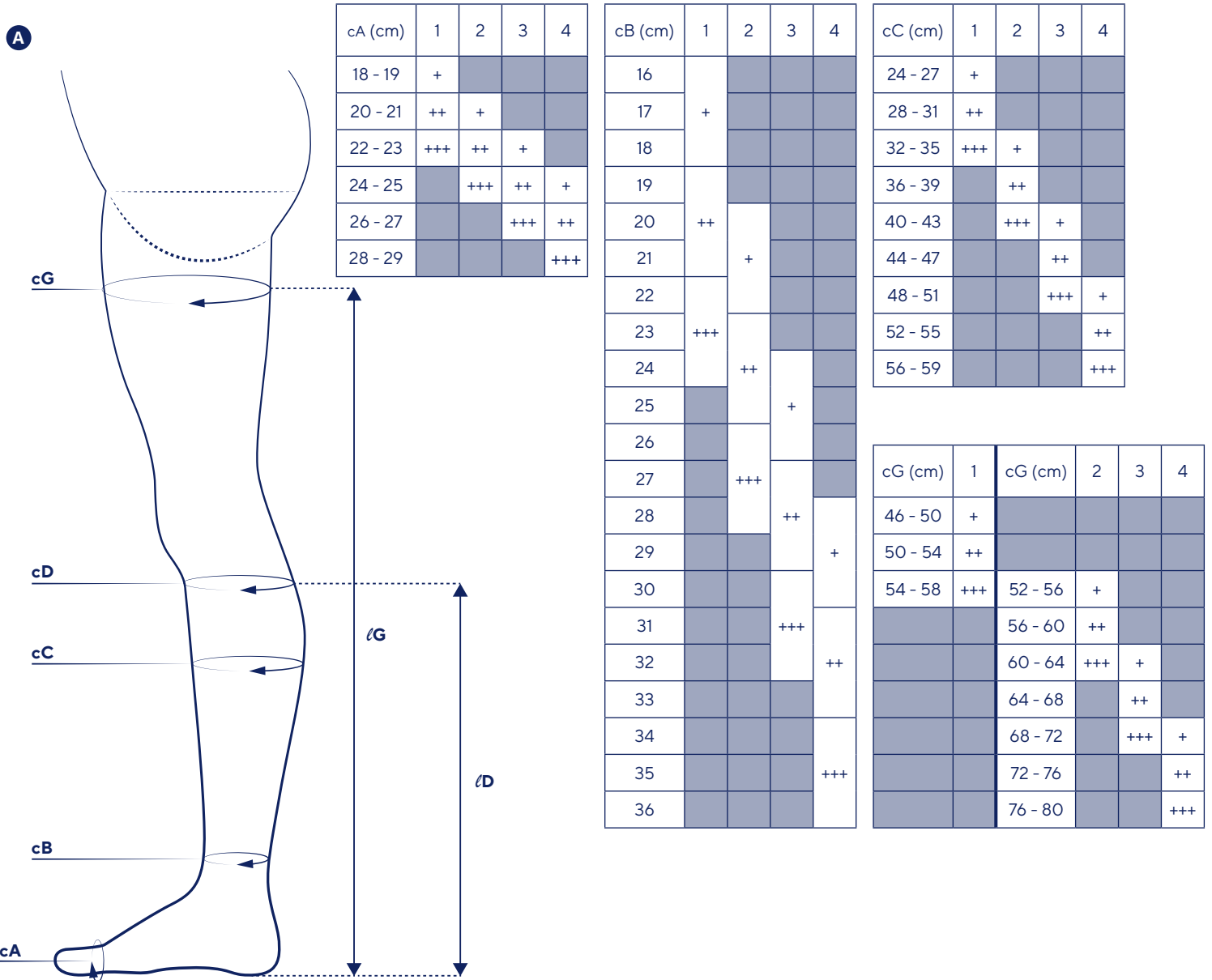
fr Vêtements mobilisateurs ajustables pour les œdèmes.....	8
en Adjustable mobilizing garments for edema.....	8
de Verstellbar mobilisierende Kleidung zur Behandlung von Ödemen.....	9
nl Verstelbare mobilisatiekleding voor oedemen.....	9
it Indumenti di mobilizzazione regolabili per edemi.....	10
es Prendas movilizadoras ajustables para edemas.....	10
pt Roupas de mobilização ajustáveis para os edemas.....	11
da Mobiliserende og justerbare beklædningsgenstande til ødemer.....	11
fi Säädettävät mobilisoivat vaatteet turvotukseen.....	12
sv Justerbara mobiliserande plagg för ödem.....	13
el Ρυθμιζόμενα ενδύματα κινητοποίησης για τα οιδήματα.....	13
cs Nastavitelné mobilizační prádlo pro léčbu edémů.....	14
pl Regulowana odzież mobilizująca na obrzęki.....	14
lv Regulējams mobilizējošs apģērbs tūskām.....	15
lt Poodinę cirkuliaciją skatinantys reguliuojami drabužiai ištinusioms vietoms.....	15
et Nahaaluseid kudesid mobiliseerivad kohandatavad riideesemed ödeemide puhuks.....	16
sl Prilagodljiva mobilizacijska oblačila za edeme.....	16
sk Nastaviteľné mobilizačné návleky na liečbu lymfedému.....	17
hu Állítható mobilizáló ödémacsökkentő öltözet.....	17
bg Регулируеми мобилизираци дрехи за отоци.....	18
ro Îmbrăcămintele de mobilizare ajustabilă pentru edeme.....	18
ru Мобилизующая одежда регулируемого размера для лечения отеков.....	19
hr Prilagodljivi odjevni predmeti za imobilizaciju kod edema.....	20
zh 用于水肿治疗的活动治疗衣.....	20
ja リンパ浮腫用調整可能活性化衣服.....	21
ko 조절 가능한 부종 모빌라이징 가먼트.....	21
ar أغطية مُحركة للوذمات قابلة للتعديل.....	22

fr	Chaussette	nl	Kous	pt	Meia até ao joelho	sv	Knästrumpa	pl	Skarpetka	et	Sokk	hu	Zokni	ru	Носок	ja	靴下
en	Below-knee	it	Calza corta	da	Strømpe	el	Κάλτσα	lv	Īsā zeķe	sl	Nogavica	bg	Чорап	hr	Čarapa	ko	양말
de	Wadenstrumpf	es	Calcetin	fi	Sukka	cs	Podkolenka	lt	Kojinė	sk	Lýtkový návlek	ro	Șosetă	zh	袜子	ar	جوارب

	cB (cm)	cC (cm)	cD (cm)	ℓD (cm)			
1	16 - 24	24 - 35	25 - 36	S 38 - 40	N 40 - 42	L 42 - 44	XL 44 - 46
2	20 - 28	32 - 43	32 - 43				
3	24 - 32	40 - 51	39 - 50				
4	28 - 36	48 - 59	46 - 57				

fr	Bas-cuisse	nl	Dijkous	pt	Meia até à coxa	sv	Lårstrumpa	pl	Pończocha	et	Reieosa	hu	Combfix	ru	Чуллок	ja	ロングストッキング
en	Thigh-high	it	Calza alla coscia	da	Lårstrømpe	el	Κάλτσα ριζομηρίου	lv	Garā zeķe	sl	Nadkolenka	bg	Чорап за бедрото	hr	Čarapa do visine bedra	ko	허벅지 스타킹
de	Oberschenkelstrumpf	es	Media-muslo	fi	Reisisukka	cs	Stehenní punčocha	lt	Kojinė iki klubo	sk	Stehenný návlek	ro	Ciorapi	zh	长筒袜	ar	أسفل الفخذ

	cB (cm)	cC (cm)	cG (cm)	ℓG (cm)			
1	16 - 24	24 - 35	46 - 57	S 67 - 72	N 72 - 77	L 77 - 82	XL 82 - 87
2	20 - 28	32 - 43	53 - 65				
3	24 - 32	40 - 51	61 - 72				
4	28 - 36	48 - 59	68 - 80				

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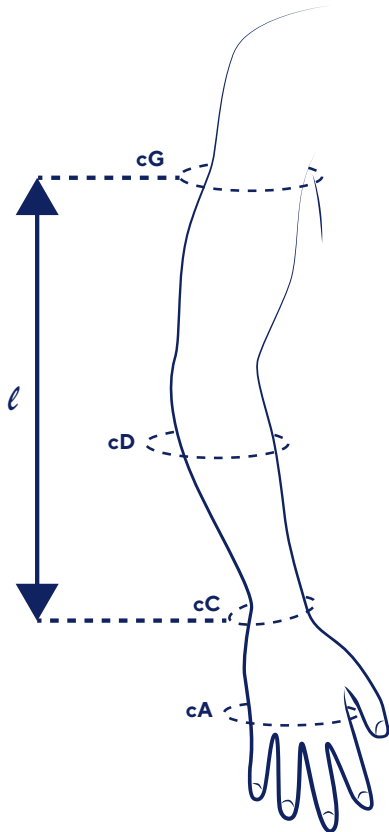
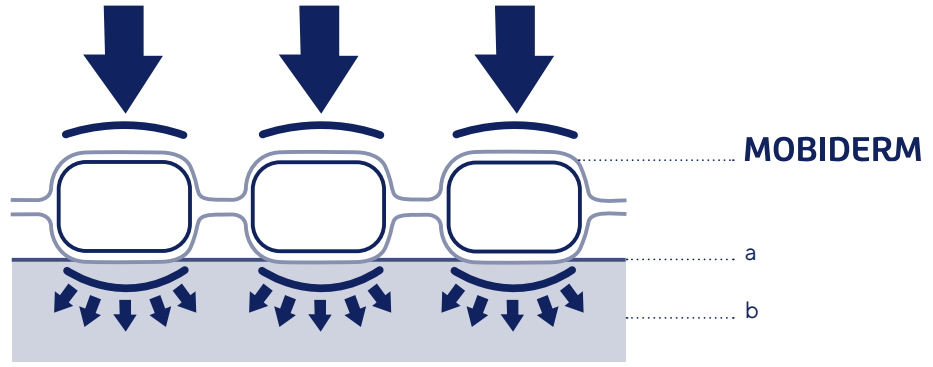
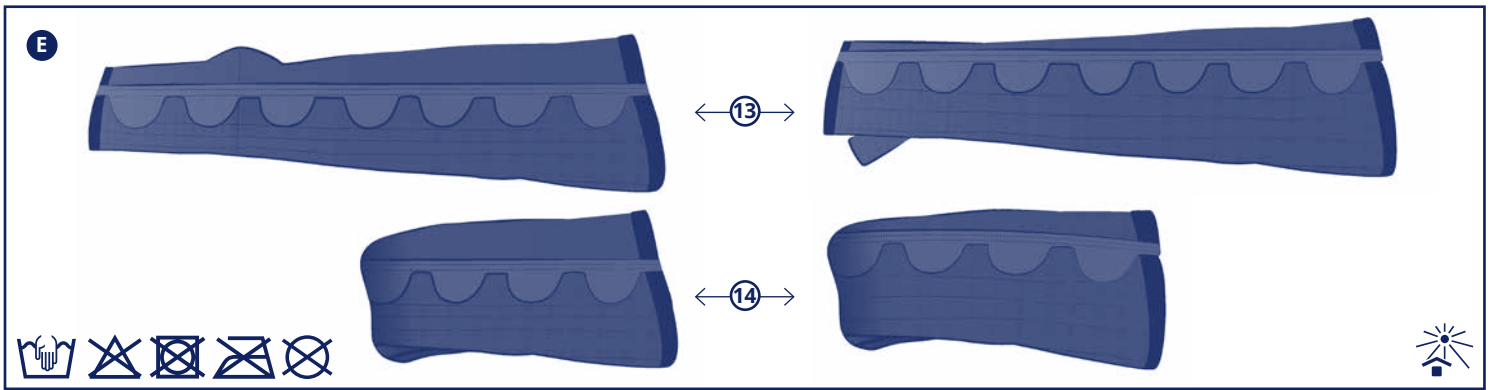
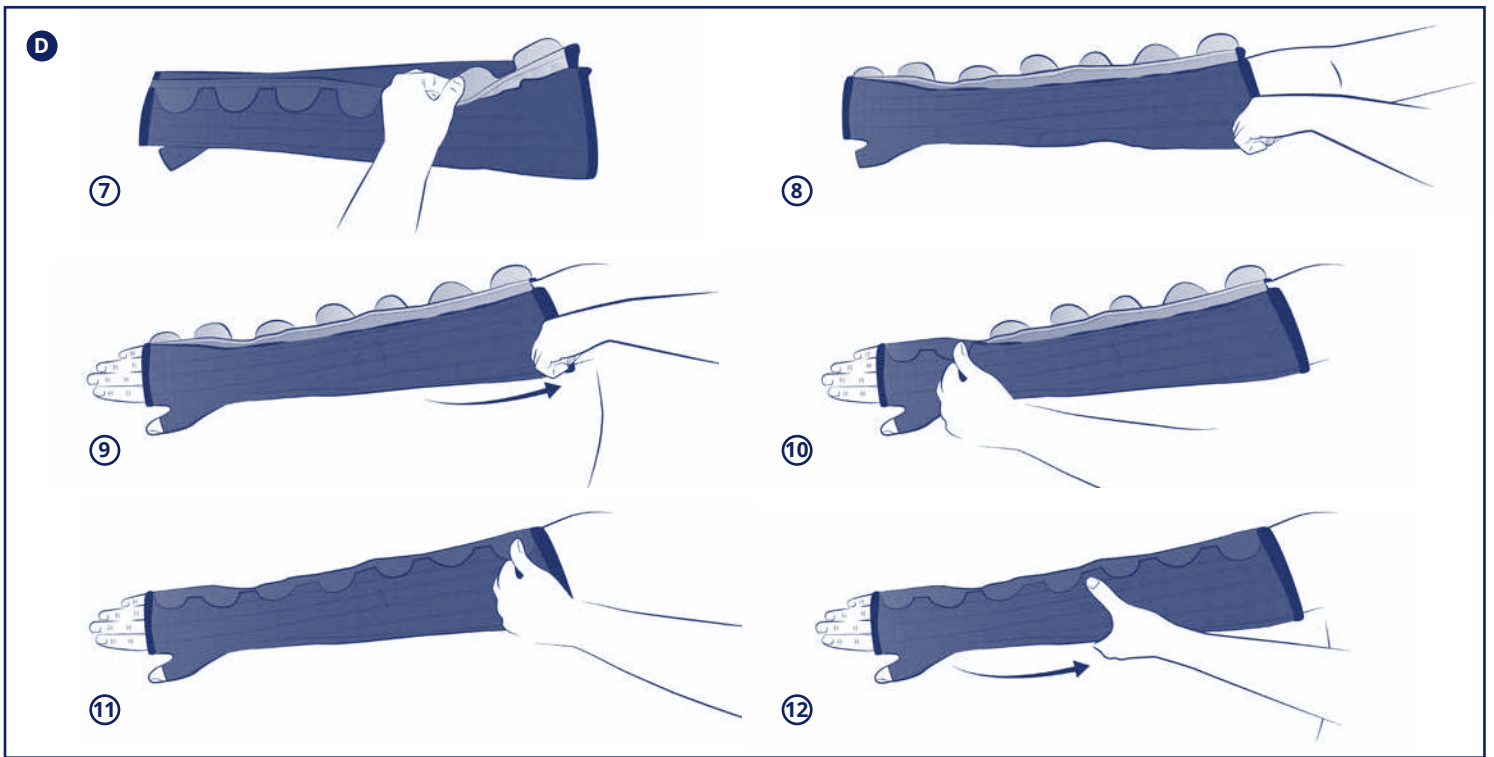
- If you suffer from advanced venous disease or ulcers, consult a health care professional for advice on a suitable full-leg compression garment.
- For reasons of hygiene, safety and performance, do not share your garment with anyone else. The product should not be reused for another patient.

Undesirable side-effect

- Skin reactions: blisters, itching, marks, burns and redness.
- Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Care/Maintenance ⓘ

To ensure your MOBIDERM AUTOFIT garments last as long as possible and to prevent infection, it is important to wash them every day, or after each use.

Close the hook-and-loop fasteners before washing.

Hand wash: soak the device in cold soapy water, rub with a clean sponge and squeeze out excess water. Air dry, away from direct heat (e.g. radiator, sun, etc.). Do not tumble-dry. Do not iron.

In a washing machine:

- Use the wool cycle at a cool temperature.
- Use a laundry detergent without fabric softener.
- Do not use chlorine-based detergents or harsh products.
- Place the garment in a mesh laundry bag to prevent damage by other garments (particularly with self-fastening tabs or zippers).
- Squeeze out by hand.
- Air dry, away from direct heat (e.g. radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

When the garment is not being worn, preferably keep it flat, with the hook-and-loop fasteners closed ⓘ. If it has to be folded, position the hooks on the outside ⓘ.

Disposal

Check local regulations for how to dispose of MOBIDERM AUTOFIT garments.

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©Thuasne - 2017801 (2020-10)