
**THUASNE**

# LombaFirst





<b>fr</b>	Ceinture de soutien lombaire.....	6
<b>en</b>	Lumbar support belt.....	6
<b>de</b>	LWS-Stabilisierungsothese.....	7
<b>nl</b>	Lumbale rugbrace.....	8
<b>it</b>	Cintura di sostegno lombare.....	9
<b>es</b>	Faja de sujeción lumbar.....	10
<b>pt</b>	Cinta de suporte lombar.....	11
<b>da</b>	Rygstøttebælte.....	12
<b>fi</b>	Lannetukivyö.....	13
<b>sv</b>	Ryggstöd för ryggen.....	13
<b>el</b>	Ζώνη οσφυϊκής υποστήριξης.....	14
<b>cs</b>	Bederní podpurný pás.....	15
<b>pl</b>	Podtrzymujący pas lędźwiowy.....	16
<b>lv</b>	Muguras jostas daļas atbalsta josta.....	17
<b>lt</b>	Palaikomasis juosmens diržas.....	18
<b>et</b>	Nimmepiirkonna tugikorsett.....	18
<b>sl</b>	Oporni križni pas.....	19
<b>sk</b>	Podporný driekový pás.....	20
<b>hu</b>	Ágyéki fűző.....	21
<b>bg</b>	Колан за поддържане на кръста.....	22
<b>ro</b>	Centură de susținere lombară.....	23
<b>ru</b>	Опорный поясничный корсет.....	24
<b>hr</b>	Lumbalni potporni pojas.....	25
<b>zh</b>	加強型腰部穩定护带.....	25
<b>ar</b>	حزام لدعم أسفل الظهر.....	27

	1	60 - 80 cm
	2	75 - 95 cm
	3	90 - 110 cm
	4	105 - 125 cm
	5	120 - 140 cm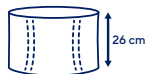

<b>fr</b>	Tour de taille	Hauteurs : 21 cm - 26 cm
<b>en</b>	Waist measurement	Heights: 21 cm - 26 cm
<b>de</b>	Taillenumfang	Höhen: 21 cm - 26 cm
<b>nl</b>	Tailleomvang	Hoogtes: 21 cm - 26 cm
<b>it</b>	Circonferenza vita	Altezze: 21 cm - 26 cm
<b>es</b>	Contorno de cintura	Alturas: 21 cm - 26 cm
<b>pt</b>	Perímetro de cintura	Alturas: 21 cm - 26 cm
<b>da</b>	Taljemål	Højde: 21 cm - 26 cm
<b>fi</b>	Vyötärön ympärys	Korkeudet: 21 cm - 26 cm
<b>sv</b>	Midjemått	Höjd: 21 cm - 26 cm
<b>el</b>	Περιμετρος μέσης	Ύψη: 21 εκ. - 26 εκ
<b>cs</b>	Obvod pasu	Výšky: 21 cm - 26 cm
<b>pl</b>	Obwód w talii	Wysokości: 21 cm - 26 cm
<b>lv</b>	Vidukļa apkārtmērs	Augstumi: 21 cm - 26 cm
<b>lt</b>	Juosmens apimtis	Aukštis: 21 cm - 26 cm
<b>et</b>	Vööümbmõõt	Kõrgused: 21 cm - 26 cm
<b>sl</b>	Obseg pasu	Višina: 21 cm - 26 cm
<b>sk</b>	Obvod pásu	Výšky: 21 cm - 26 cm
<b>hu</b>	Derék bősége	Magasság: 21 cm - 26 cm
<b>bg</b>	Обиколка на талията	Височини: 21 cm - 26 cm
<b>ro</b>	Circumferința taliei	Înălțimi: 21 cm - 26 cm
<b>ru</b>	Обхват талии	Высота: 21 см - 26 см
<b>hr</b>	Opseg struka	Visine: 21 cm - 26 cm
<b>zh</b>	腰围	高度 : 21 cm-26 cm
<b>ar</b>	محيط الخصر	الارتفاعات: 21 سم - 26 سم

<b>fr</b>	Compression	Maintien	Education posturale
<b>en</b>	Compression	Support	Postural education
<b>de</b>	Kompression	Halt	Erlernen der Körperhaltung
<b>nl</b>	Compressie	Fixatie	Voorlichting over houding
<b>it</b>	Compressione	Sostegno	Educazione posturale
<b>es</b>	Compresión	Sujeción	Educación postural
<b>pt</b>	Compressão	Manutenção	Educação postural
<b>da</b>	Kompression	Støtte	Postural genoptræning
<b>fi</b>	Kompressio	Tuki	Asentokuntoutus
<b>sv</b>	Kompression	Stöd	Postural träning
<b>el</b>	Συμπύεση	Συγκράτηση	Σωστή στάση σώματος
<b>cs</b>	Komprese	Stabilizace	Správné držení těla
<b>pl</b>	Kompresja	Podtrzymywanie	Edukacja dotycząca postawy
<b>lv</b>	Kompresija	Atbalsts	Stājas korekcija
<b>lt</b>	Kompresija	Palaikymas	Laikysenos taisymas
<b>et</b>	Kompressioon	Hooldus	Rühitreening
<b>sl</b>	Kompresija	Oprjem	Učenje telesne drže
<b>sk</b>	Kompresia	Podpora	Cvičenie posturálnych svalov
<b>hu</b>	Kompresszió	Támasz	Tartásjavító oktatás
<b>bg</b>	Компресия	Поддръжка	Приучаване към правилна стойка
<b>ro</b>	Compressie	Susținere	Educație posturală
<b>ru</b>	Компрессия	Стабилизация	Исправление осанки
<b>hr</b>	Kompresija	Potpoma	Učenje o držanju
<b>zh</b>	压紧	稳定	姿势联系
<b>ar</b>	الضغط	الدعم	تدريب وضعي

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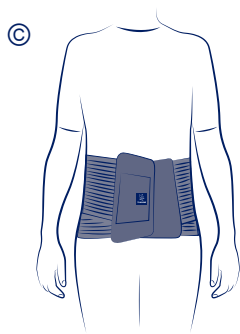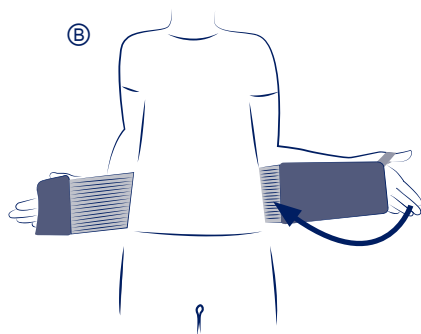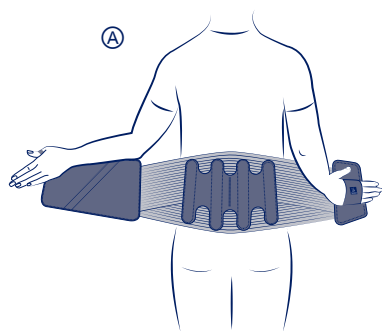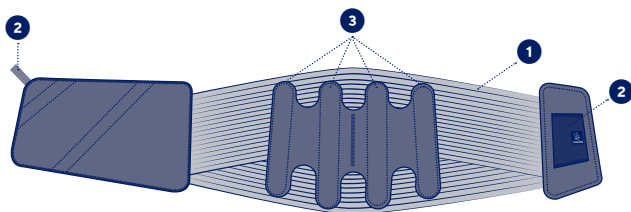## LUMBAR SUPPORT BELT

### Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

- 1 Elastic fabric.
  - 2 Abdominal panels with self-fastening surface, finger-loop and thumb-loop.
  - 3 Conformable dorsal stays.
- Textile components: polyamide - polyurethane - polyester - elastane.  
Non-textile components: brushed tempered steel - polyoxyméthylène.

### Properties/Mode of action

The device relieves low back pain, supporting the lumbar spine: compartmental lumbar unloading is therefore achieved via increased intra-abdominal pressure.

### Indications

Non-specific low back pain (acute, subacute and chronic phases).  
Spinal stenosis.

### Contraindications

Do not use in the event of known allergy to any of the components.  
Do not use on pregnant women.  
Do not use in the event of a hiatus hernia.  
Do not use in the event of bone cancer with spinal metastases.  
Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

### Precautions

Verify the product's integrity before every use.  
Wear the device above thin clothing.  
Do not use the device if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart.  
Strictly comply with your healthcare professional's prescription and recommendations for use.  
It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.  
Do not wear the product in a medical imaging machine.  
Do not wear the product while sleeping.  
Consult a health professional if discomfort or pain occurs.  
The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

### Undesirable side-effects

This device can cause skin reactions (redness, itching, irritations...).  
Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Position the belt around the waist, placing your hands in the finger loop and thumb loop located on the abdominal panels. ④  
The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine. The bottom of the belt must stop at the top of the crease between the buttocks.  
The Thuasne logo indicates which way up the belt should be put on.

Pull the finger loop and the thumb loop on the abdominal panels simultaneously and evenly. ⑤  
Fasten the belt, right side over left side, adjusting to the desired compression. ⑥  
The level of compression can be adjusted as required throughout the day.

### Care

Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.  
Keep this instruction leaflet

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©Thuasne - 2124202 (2021-07)